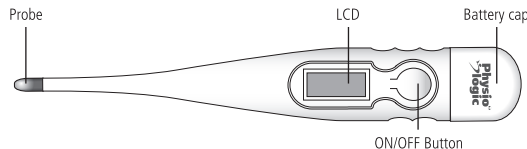
Digital 10™

Thermometer / Thermomètre

PLEASE READ CAREFULLY BEFORE USING.

This digital thermometer is intended to measure the human body temperature orally, rectally or under the arm, and can be used on people of all ages. To better understand its functions and to provide years of dependable results, please read all instructions first.

This appliance conforms to the following standards: EN 12470-3, ISO 80601-2-56, IEC 60601-1-11, complies with the requirements of IEC 60601-1-2(EMC), AAMI/ANSI ES60601-1(Safety) standards, and the manufacturer is ISO 13485 certified.

CLEANING INSTRUCTIONS

Before and after each use, clean the thermometer with a soft, damp cloth. Finish by wiping with a soft dry cloth. For disinfection, 70% Ethanol or Isopropyl alcohol can be used. Do not immerse any part of the thermometer in alcohol or hot water above 50 °C (122 °F). The use of any other solution or cleaning method beyond the above mentioned may result in reliability problems with the thermometer and void its warranty.

HOW TO SWITCH BETWEEN °C AND °F

When the unit is turned off, press and hold the on/off button for four (2) seconds. When the desired measurement scale is flashing (°C or °F), it is ready for use.

NOTES:

- Use of a probe cover may result in a 0.1 °C (0.2 °F) discrepancy from actual temperature.
- DO NOT use this thermometer in the ear. DO NOT use it orally on children two years or younger. Do not use it orally if it has been used rectally.
- When taking a temperature, do not remove the thermometer until beeps indicate that measuring is complete. The unit will sound long beeps unless the temperature detected is 37.8°C (100°F) or higher, in which case it will sound groups of 3 fast beeps to signal a possible fever.

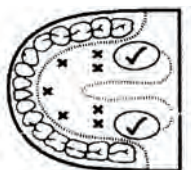
HOW TO TAKE YOUR TEMPERATURE

1. Decide whether the measurement will be oral, underarm or rectal.
2. Press On/Off button. When the °C (°F) symbol starts flashing (this takes a few seconds), the thermometer is ready for use.

3.1 Taking an oral temperature

NOTE: Drinking hot or cold fluids, exercising, smoking or performing other activities may raise or lower body temperature. It is important to relax for approximately 5 minutes, mouth closed prior to taking a reading.

1. Place the probe tip under the tongue as near as possible to the heat pocket, close your mouth and avoid moving the unit. Your mouth should remain closed until beeps are heard to indicate that temperature reading is complete.

3.2 Taking an axillary temperature (under the arm)

NOTE: results with this method are usually 0.5°C - 1°C lower than if measured orally.

1. Wipe armpit with dry towel then place the probe tip under the arm so the tip is touching skin, with the thermometer perpendicular to the body. Position the arm across the chest so the probe tip is well covered by the arm. This also ensures that the probe is not affected by the room's air. Beeps will sound to indicate that temperature reading is complete.

3.3 Taking a rectal temperature

NOTE: Rectal temperatures are an appropriate and reliable method for infants and small children. Otherwise, this method should only be used when it is impossible or impractical to take an oral or axillary temperature.

1. Apply a water-soluble lubricant to the tip. Insert the probe into rectum MAXIMUM 1 cm (½") until beeps indicate that temperature reading is complete.
4. The power will be shut off automatically after approximately 10 minutes.

GENERAL CARE

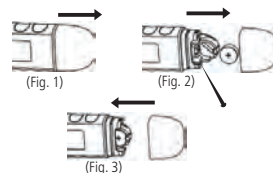
1. This thermometer should only be used under the supervision of an adult.
2. Do not walk, run or talk while using the thermometer.
3. Clean the thermometer as directed before and after each use.
4. Store the unit in the protective case when not in use.

5. Do not bite the probe.
6. Do not store the unit where it will be exposed to direct sunlight, dust or humidity. Avoid extreme temperatures.
7. Dropping or subjecting your thermometer to strong shocks should be avoided.
8. Do not attempt to disassemble the unit, except to replace the battery.

CHANGING THE BATTERY

The battery in your digital thermometer needs to be replaced when the symbol  appears in the middle right of the display. Replace the battery with a 1.55 Vdc battery, type SR41, LR41, UCC392 or equivalent.

1. Slide the battery cap away from the unit. (Fig.1)
2. Gently pull out plastic circuit board with battery chamber approx. 1 cm (± ½ inch). (Fig. 2)
3. Using a non-metal pointed tool, carefully remove the battery and insert the new battery with the positive (+) side up. (Fig. 2)
4. Carefully slide the battery cap back into place, making sure that the cap fits snugly to protect the battery compartment from moisture. (Fig.3)


Note: Please properly dispose of the battery away from small children and heat.

SPECIFICATIONS

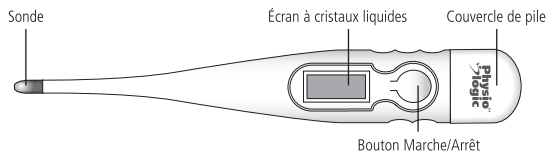- Type: Digital thermometer (non predictive)
- Measuring range: 32.0°C – 42.9°C (90°F – 109.9°F)
- Accuracy: ±0.1°C (±0.2°F) for readings between 35.5°C~42.0°C (95.9°F~107.6°F) at 18°C~28°C (64.4°F~82.4°F) ambient temperature. In other conditions: ±0.2°C (±0.4°F)
- Operating mode: Direct mode
- Display: Liquid crystal display, 3 1/2 digits
- Memory: Last stored value
- Battery: One 1.5 V DC. button battery (size LR41 or SR41, UCC 392)
- Battery life: Approx. 200 hours
- Dimensions: 12.2 cm x 1.8 cm x 1.1 cm (L x W x H)
- Weight: Approx. 10 grams including battery
- Ambient operating range: Temperature: 5°C~40°C (41°F~104°F) Relative humidity: 15%~95%
- Storage and transportation conditions: Temperature: -20°C~55°C (-4°F~131°F) Relative humidity: 15%~95%
- Classification: Type BF

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