


Physio
Logic[®]

Now you know
Soyez sûr[™]

Fever Bugz[®]

Stick-On Fever Indicators
Timbres thermo-indicateurs

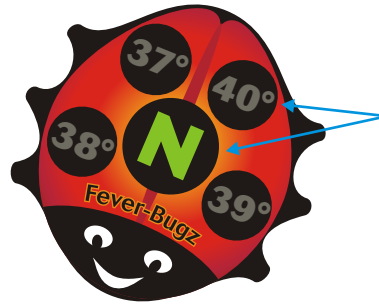

Child's Temperature
appears inside the bug's
black spots

016-700 - Instructions

Peel off from the backing and apply directly to the skin. For optimal accuracy place under the armpit, on the forehead or on the chest. Within seconds Fever-Bugz light-up indicating N (for normal), 37°, 38°, 39°, or 40° (and over). The temperature reading corresponds to the highest figure that appears green.

Normal axillary temperature range is 36.1°C to 37.2°C. Fever-Bugz will adhere through bathing and may be left in place for up to 48 hours on normal, healthy intact skin. Removing and replacing reduces adhesion time. To Remove: Gently peel off and discard. DO NOT REUSE.

- Continuous Monitoring
- Stick-on up to 48 hrs
- Fast, safe and effective
- Kids love to wear them
- Hypo allergenic
- Accurate within 0.5° C

Physio
Logic[®]

To Learn more about what Fever Bugz[®], Stick-on Fever Indicators can do for you,
contact your AMG Sales Representative or, call: (toll free) 1-800-363-2381 (Montreal) 514-737-5251

AMG, Physio Logic, and Fever Bugz are trademarks of AMG Medical Inc.