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**www.proactive-painmanagement.com**

## INSTRUCTIONS: EYE MASK

Mask should be applied for 20 minutes, or as recommended by a doctor. Remove and wait at least 20 minutes. Repeat as needed. To further reduce hot or cold level, place mask in soft cloth covering before applying.

### For Heat Therapy:

Microwave mask per the below suggested heating times and test mask temperature. Heat for additional 3 seconds as needed until desired temperature is reached.

#### Suggested heating time for one (1) mask:

700 Watts 15 seconds	1000 Watts 12 seconds	1250 Watts 10 seconds
-------------------------	--------------------------	--------------------------

#### Do not exceed heating time for one (1) mask:

700 Watts 21 seconds max.	1000 watts 18 seconds max.	1250 Watts 16 seconds max.
------------------------------	-------------------------------	-------------------------------

### For Cold Therapy:

Freeze for at least 2 hours in a freezer storage bag.

### Caution:

Always test eye mask temperature prior to use. Overheating can cause damage or destroy mask. Prolonged use may cause injury to user including burns. For external use only. Do not use on infants. Never sit, lie, or lean against the mask. Do not puncture mask or ingest contents. Do not apply to sensitive/insensitive skin or open wounds. Use only for intended purposes. Do not fall asleep using mask. Use only under adult supervision. For chronic, extreme or prolonged pain, please contact your physician.

### Storage:

Always keep mask in a freezer storage bag when kept in the freezer.

### Cleaning:

To clean mask, use mild soap and water and air dry before reusing.

Manufactured and printed in China for:

**A.M.G. Medical Inc.**

Montreal, QC H4T 1V5 1-800-363-2381 West Chazy, NY 12992 1-888-412-4992

**www.proactive-painmanagement.com**

Rev 1 0417 P/N 911-740800