## Precautions

1. Ensure the toilet seat is in the down position before use.  
Not doing so may cause damage to commode and possible injury.
2. Make sure to read and understand these assembly instructions before installing the commode.  
If you cannot understand these instructions, contact a healthcare provider or technical professional before you attempt to install this product.
3. Inspect all parts for shipping damage before assembly. If there is shipping damage, do not use the product.  
Contact the provider/carrier for further instructions.
4. In order to ensure that the leg extensions are securely locked in position, make certain that the "E" Clips protrude through the hole of each leg extension.
5. Ensure all screws, nuts and/or bolts are tightened.
6. The footrests can be folded out of the way when they are not in use by folding the footplate upward, releasing the locking mechanism, and then folding the footplates inward towards the front of the chair.
7. Users with limited physical strength should be supervised or assisted while using commodes.
8. The arm supports are meant for assistance only, **Do Not** attempt to use the arms to support full-body weight.
9. The arms should be pushed back when transferring onto or off of the chair.

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## Lifetime Limited Warranty

Your **MedPro Defense**® commode is warranted to be free of manufacturer's defect of material and workmanship for the lifetime of the commode to the original consumer purchaser.

This warranty does not cover product failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your **MedPro Defense**® product or this warranty, please contact A.M.G. Medical.

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To learn more about this or any other MedPro Defense product, please contact your **A.M.G. Medical Inc.** representative or call toll-free: **1-800-363-2381**.

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**A.M.G. Medical Inc.**  
Montréal, Québec  
H4T 1V5 Canada  
www.amgmedical.com

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