BASIC PRECAUTIONS

1. Before assembling and/or using the commode, please read and be sure to clearly understand this instruction sheet. Failure to do so could result in personal injury and/or damage to the commode.
2. Examine the commode for any damages that may have occurred during shipping. Do not use the commode if there is apparent damage. Please contact your dealer for repairs and/or for information on how to return the product.
3. Make sure that the toilet seat is at room temperature before installing it if it has been exposed to cold. Failure to do so could result in the breakage of the commode and/or possible injury.
4. **Users with limited physical capability should be supervised and/or assisted while using the commode.**

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MAINTENANCE

1. We recommend that you regularly verify that all the hardware parts are properly tightened.
2. This commode can be easily cleaned with mild liquid household cleaners. Avoid the use of harsh solvents, abrasive powders or scouring pads.

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Folding Commode Chaise d'aisance pliable Silla bacín plegable

ASSEMBLY INSTRUCTIONS

IMPORTANT: Read the information and precautions on the back of this document before assembling / using this commode. If you are experiencing problems with this product, contact us first at the number provided. We will be glad to provide you with the support you need.

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Model / Modèle / Modelo: 770-309

LIFETIME LIMITED WARRANTY

Your MedPro® Folding Commode is warranted to be free from defects in material and workmanship under normal use. The warranty does not extend to non-durable components which are subject to normal wear and replacement.

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AMG Medical Inc.